


Water Supply Device for Endoscope

WATER PLEASE AF-WP1

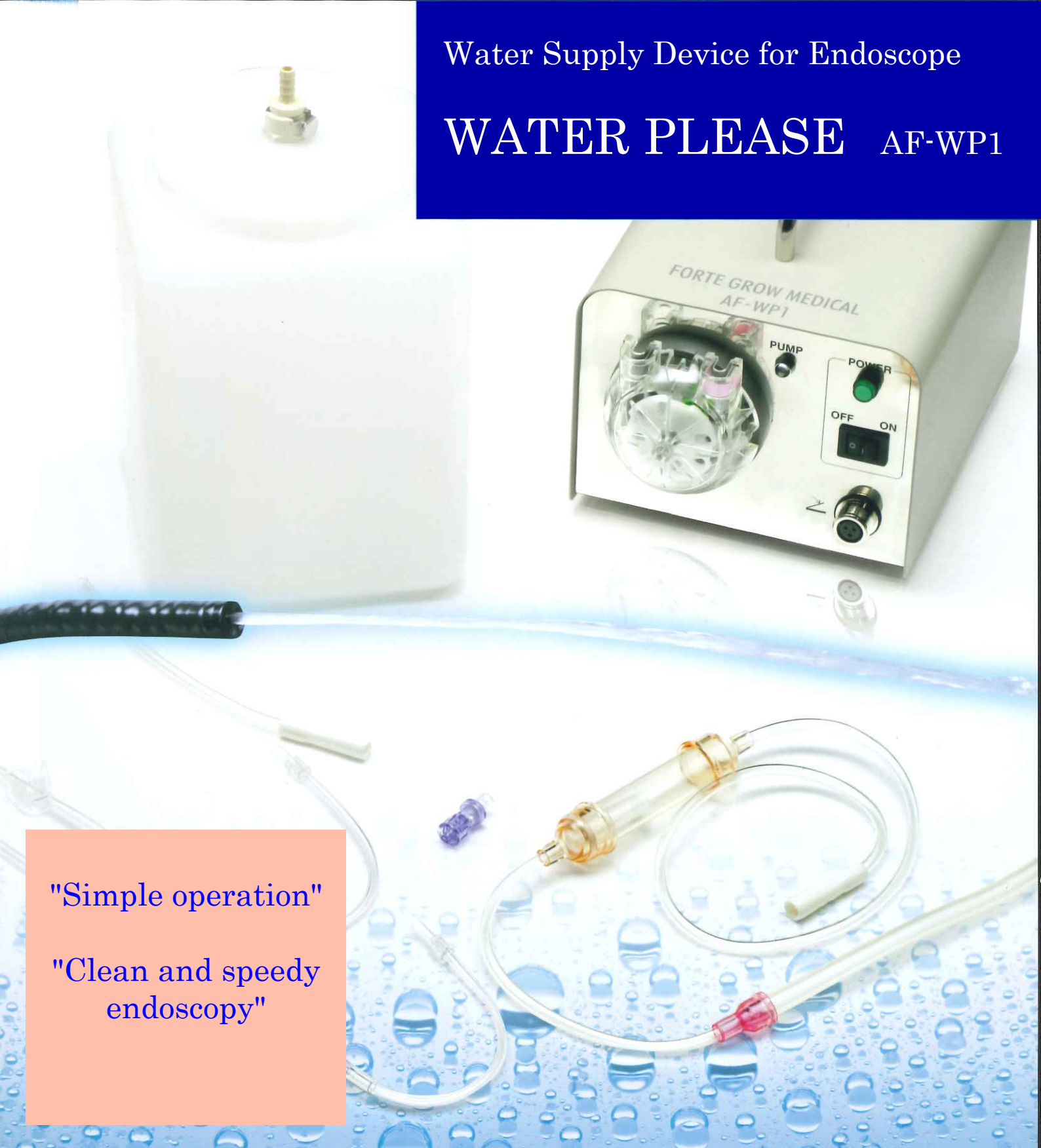

"Simple operation"

"Clean and speedy
endoscopy"

Water Tube : AF-WT, AF-WPU, AF-WV, AF-WTA

Water Tank : AF-WTA4L



FORTE GROW MEDICAL CO., Ltd.

Making endoscopy more accurate and more speedy!!
The simple operation drastically shortens an inspection time.

LESS STRESS

- Using water supply machine provides more distinct images.
Inspections are carried out efficiently and smoothly.
- It's easy to find accurately bleeding spots.
A water tank has large capacity "4 liters".
- It acquired high reputation from many users.
WP1 shortens an inspection time drastically, and lighten the burden of inspection.

HIGH EFFICIENCY


- It has accomplished simple operation by using Foot Switching.

The edge of the tube connecting with endoscope has enough strength to keep stable water flow.




Locking structure is adapted for connection to prevent from slipping out.

Maximum Water Supply
700cc/ min

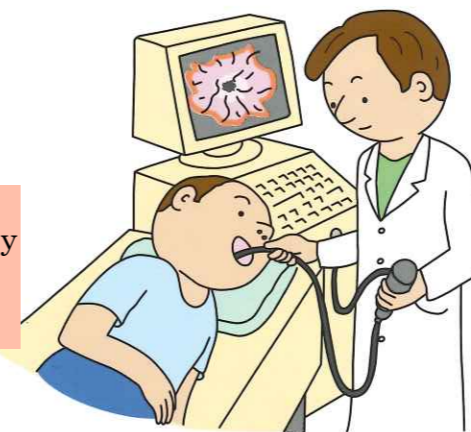


Water Please

Foreign substances will be eliminated by the attached filter.
*Filter removal capacity is limited.

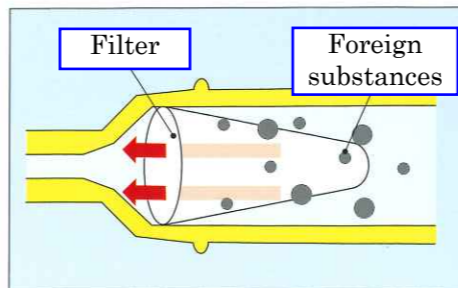

Non-return valve prevents from back flow which may cause an infection.

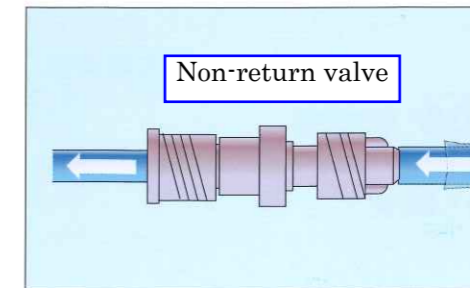

HOW TO USE



1) Attaching a pump tube (AF-WPU) to a pump cassette and put on WP1, and connecting a water tube (AF-WT) to the pump tube.



2) Putting tubes (AF-WPU, AF-WTA) on a tank with water inside



3) Putting a tube (AF-WT) on a scope



4) Cleaning spots with water using foot switching

How to use "WATER PLEASE"





WATER PLEASE AF-WP1

Specifications



WATER PLEASE AF-WP1

Flow rate: 700ml/min (±10%)
Size: 168mm(W) x 165mm(H) x 246mm(D)
Weight: 2.8kg

Included item: water tank 1 unit

Manufacture and Distribute by DKK TOA CORPORATION

WATER TANK AF-WPA4L

Capacity: 4L
Size: 152mm(W) x 210mm(H) x 152mm(D)

Manufacture and Distribute by Forte Grow Medical Co.,Ltd.



AF-WT (逆止弁一体型、フリー型)

< WATER TUBES (standard) >

AF-WT (w/ non-return valve)	approx 1,750mm	(50pcs/box)
AF-WPU (pump)	approx 2,300mm	(50pcs/box)
AF-WTA (tube for inside a tank)	approx 200mm	(50pcs/box)

Manufacture and Distribute by Forte Grow Medical Co.,Ltd.



AF-WPU (ポンプ)



AF-WV (逆止弁)




AF-WTA (タンク内チューブ)

< WATER TUBES (customized) >

AF-WT	approx 1,800mm	(50pcs/box)
AF-WV (non-return valve)	approx 30mm	(50pcs/box)
AF-WPU (pump)	approx 2,300mm	(50pcs/box)
AF-WTA (tube for inside a tank)	approx 200mm	(50pcs/box)

Manufacture and Distribute by Forte Grow Medical Co.,Ltd.

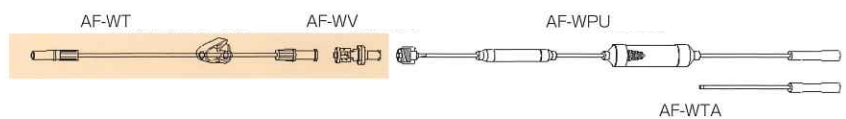

【 Precautions for use 】

- 1) make sure to read through the attachment before use.
- 2) avoid using if products have any defects or package boxes having damages.
- 3) guard against wet and direct sunlight, and not store them in high temperature and humidity places.
- 4) make sure that a scope is turned on for water supply.
- 5) be aware that water might leak from a part of rubber cap supplying water, if the rubber cap for clamp channel of scope is loose or damaged.
- 6) All water tubes are for single use.

The product specification might be changed without advanced notices. We appreciate your understanding.

Forte Grow Medical Co., Ltd.

Tokyo Sales Office:

51-5F Kandasakumagashi, Chiyoda-ku,

Tokyo 101-0026

TEL: 03-3863-1901 FAX: 03-3863-1905

<https://www.fgm.co.jp/index.html>